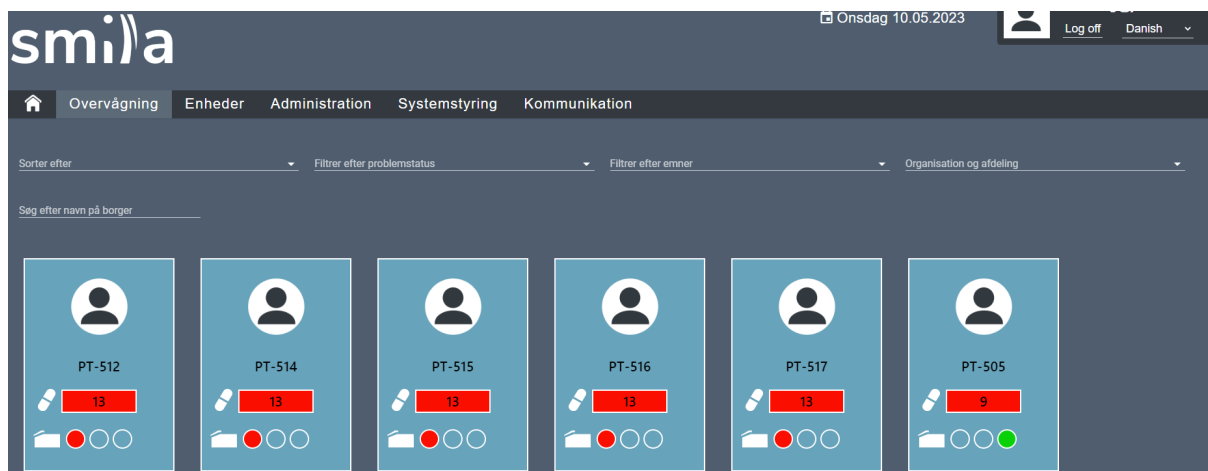
Brugervejledning SMILA Cloud

JDM Innovation GmbH

Carl-Benz-Strasse 16

DE - 71711 Murr, Germany

+49 - 7144 – 81 21 0

www.jdm.de

info@jdm.de

Indhold

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[ANNULLER](#)
[OPRET](#)

Edit patient PT-154

Photo	Photo (jpg, png, tiff)
First name*	Last name*
Birthdate*	Language*
Social Security number	Phone
Mobile phone	Country
City	Street
Zip Code	Email
Pre talking time window in minutes	Talking time window in minutes
Wider Center*	

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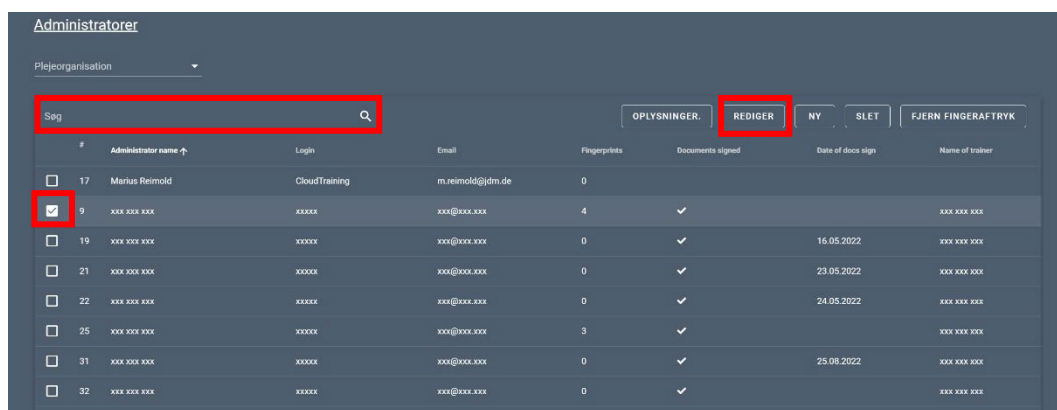
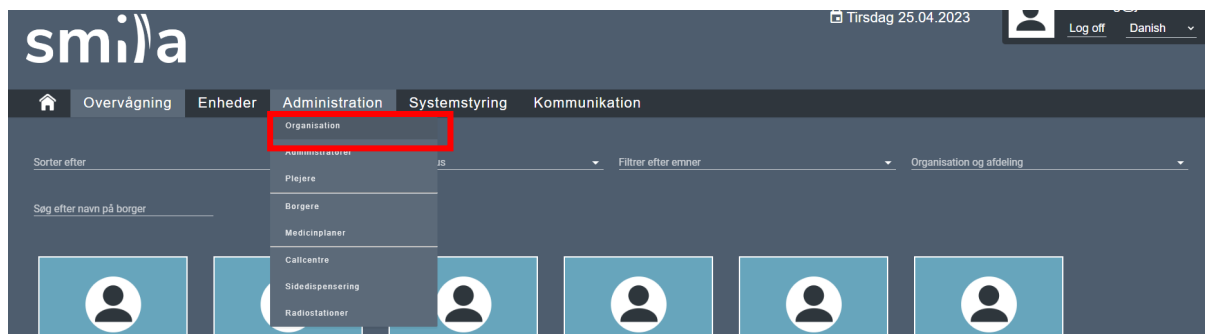
Organisation*
Specific Care Organization X

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Sprog*
English

Aktiv til kl. *
24:00

Time based alerting is only available for SMS alerts. It is possible to use email or telegram alone or in addition. This will lead to the following situation:

- Chosen channels: SMS, Telegram or/and Email. The caregiver will receive the alerts via SMS in the selected shift times. Email and Telegram alerts will be received 24/7.
- Chosen channels: Telegram or/and Email. Alerts will be received 24/7. Editing the work times has no effect.
- SMS- Alerts that are sent during non-working times, will not be received by the specific care giver.

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The screenshot displays the SMILA web application interface. At the top, the 'smila' logo is on the left, and the date 'Tirsdag 25.04.2023' and user controls ('Log off', 'Danish') are on the right. A navigation bar below the header contains links: 'Overvågning', 'Enheder', 'Administration', 'Systemstyring', and 'Kommunikation'. The main content area is titled 'Borger' and features a profile card for a patient with masked details (xxx xxx xxx, xxx xxx, Fødselsdato: 01.01.1901, Information om personen: xxx, Klient-ID: 501). To the right of the profile is a 'Historik' section showing a list of events with timestamps and technical status messages. Below the profile card are five summary boxes: 'Oprettede medicintider i cloud' (14 Poser lagt op i cloud), 'Tilgængelige poser' (14 Poser Tilgængelig), 'Poser i skuffen' (3 Poser i alt), 'Status for enheden' (Online, Enhed online), and 'Ikke taget til tiden' (31 Poser ikke taget til tiden). A bottom box shows 'Oversigt over poser' (447 Poser). On the right side of the main area, there is a box for 'Enheds-id' (Smila 00401).

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
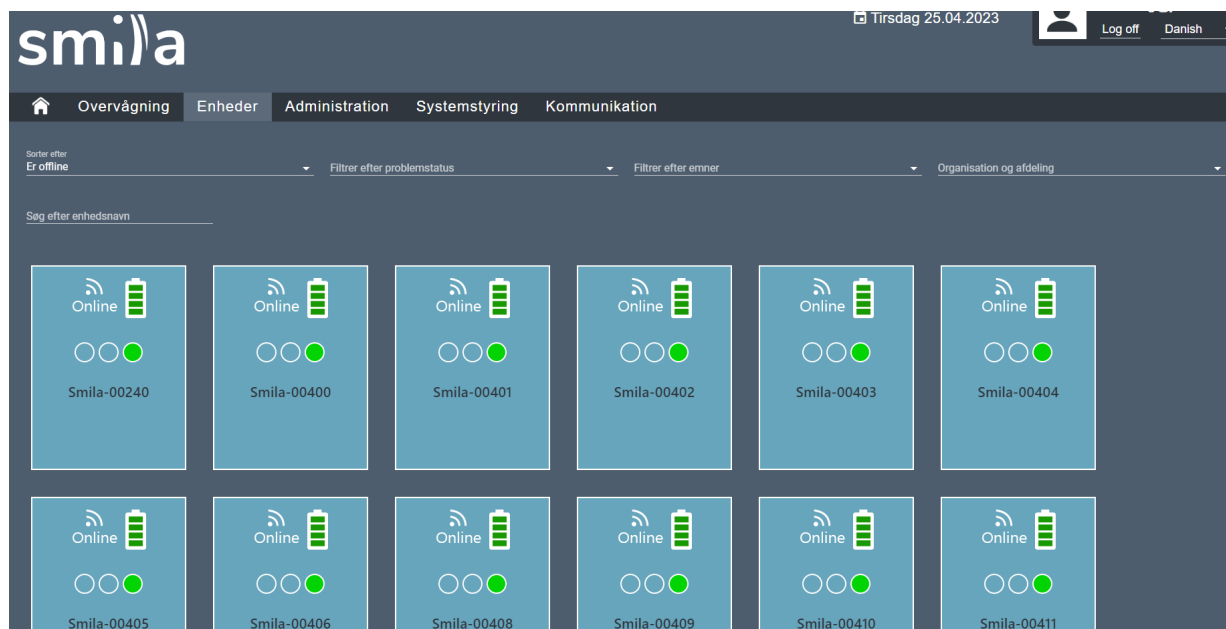
The screenshot displays the SMILA web interface for unit management. The top navigation bar includes the SMILA logo, a date indicator for Friday, 19.01.2024, and a user profile for c.mercier@jdm.de with options to log off or switch to Danish. The main menu contains links for Overvågning, Enheder, Administration, Systemstyring, and Kommunikation. The 'Enheder' section is active, showing details for unit 'Smila-00265'. This section includes a 'Placering af enhed' (Unit location) field with placeholder text, a 'Sidste check' (Last check) timestamp of 19.01.2024 14:15:05, and a 'Tilgængelige poser' (Available doses) count of 0. A 'Sidste ændring' (Last change) section lists program versions and update details. Below this, there are several status and action panels: 'Oprettede medicintider i cloud' (Created medication times in cloud) with 4 doses, 'Status for enheden' (Unit status) showing 'Online', 'Fjernbetjent indføring for' (Remote entry for) with 'LC' and 'RC' buttons, 'Foto-scan' (Photo scan) button, 'Borger-ID: 561' (Citizen ID) field, and 'TILFØJ NOTE' (Add note) button. The bottom section shows 'Poser i skuffen' (Doses in drawer) with 0 doses, 'Dispenserede poser' (Dispensed doses) with 0 doses, 'Skippede poser' (Skipped doses) with 1 dose, and 'Alle poser' (All doses) with 11 doses.

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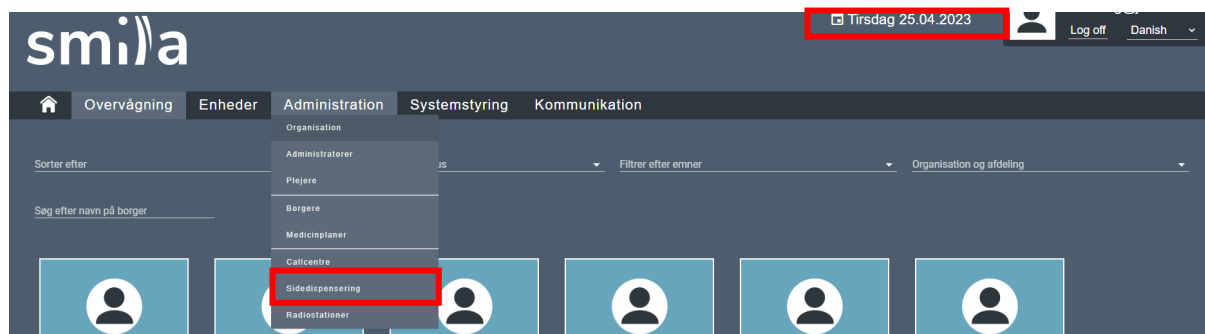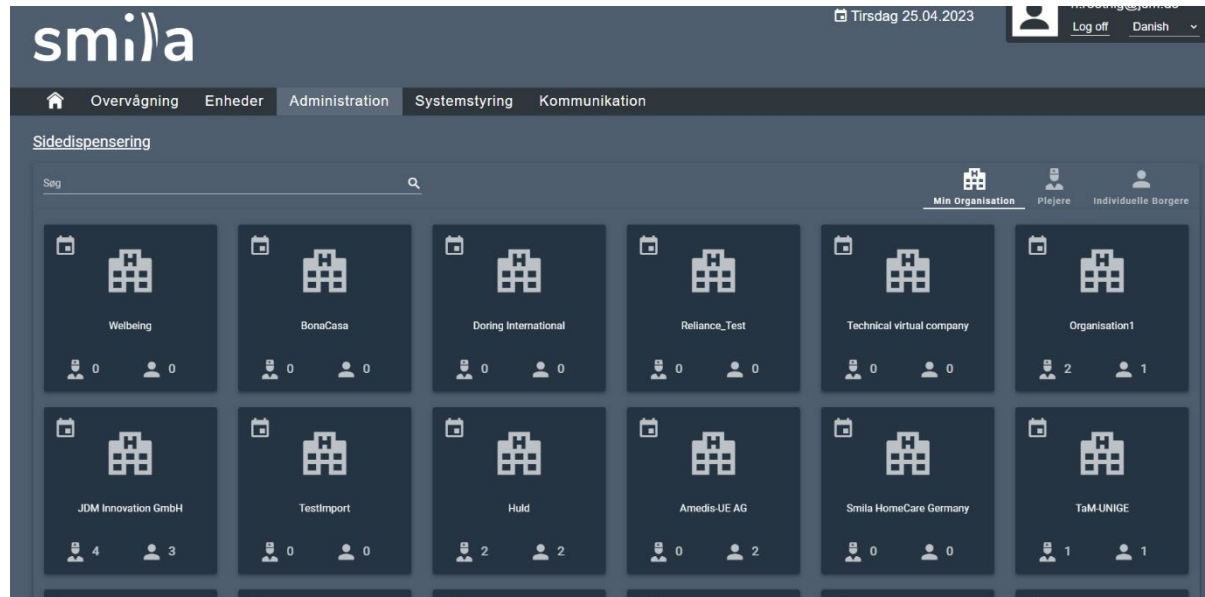
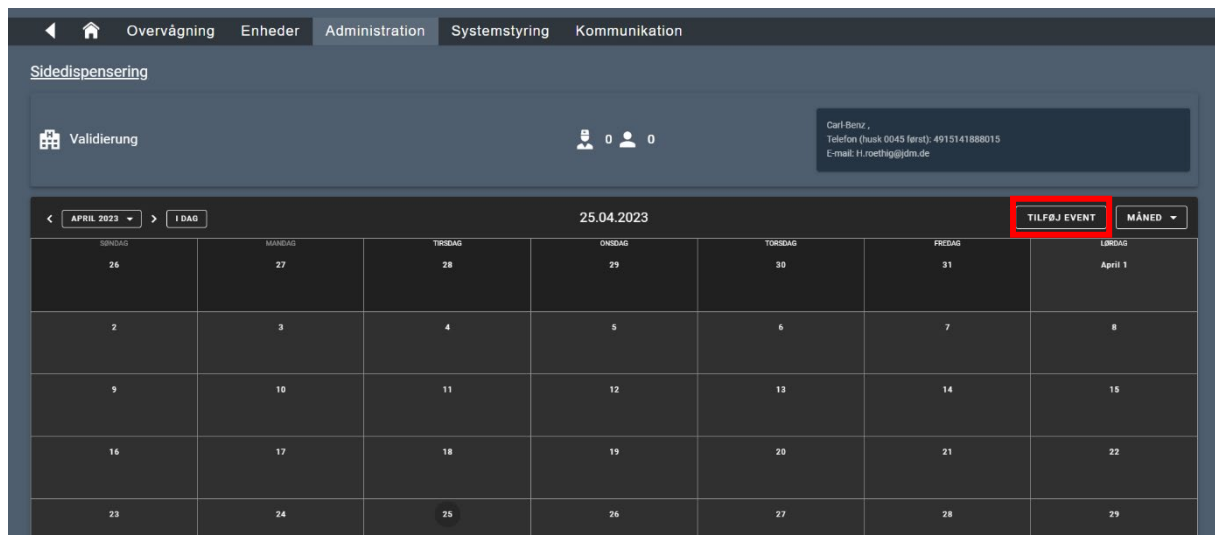
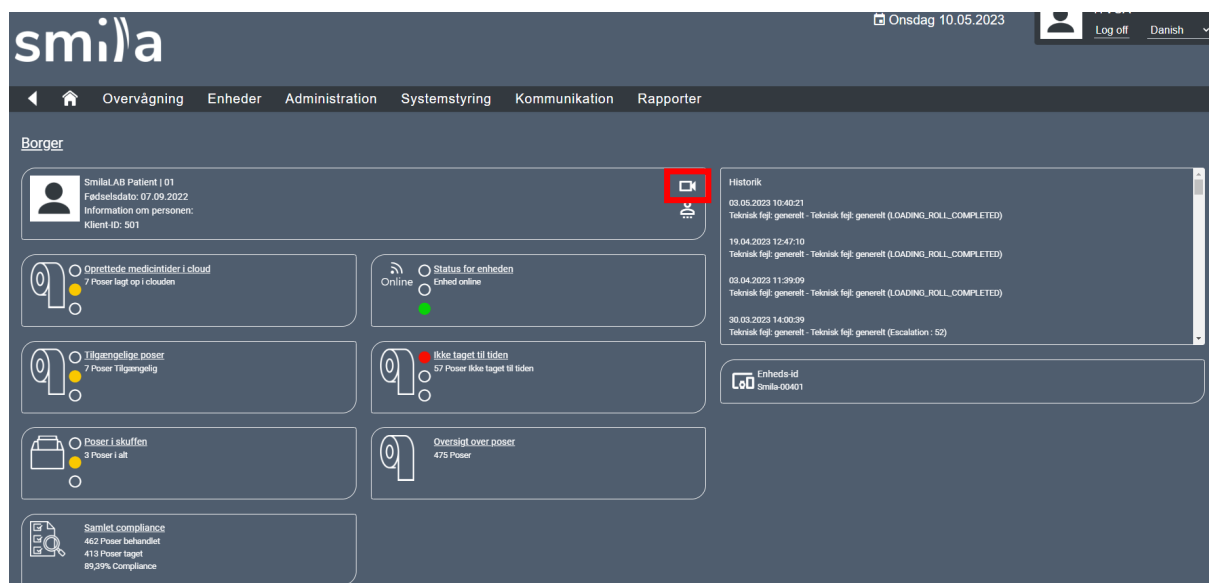
The screenshot shows the Smila Cloud interface. At the top, there is a navigation bar with the Smila logo and a date/time display: 'Onsdag 10.05.2023'. Below the navigation bar, there is a 'Borger' (Patient) section. In this section, there is a patient profile card for 'Smila AB Patient | 01' with a birth date of '07.09.2022' and a client ID of '501'. To the right of the profile card, there is a red square icon with a white camera symbol, which is the video call icon. Below the profile card, there are several status cards: 'Oprettede medicintider i cloud' (7 Poser lagt op i cloud), 'Tilgængelige poser' (7 Poser Tilgængelig), 'Poser i stuefløn' (3 Poser i alt), 'Samlet compliance' (462 Poser behandlet, 413 Poser taget, 89,39% Compliance), 'Status for enheden' (Online, Enhed online), 'Ikke taget til tiden' (57 Poser ikke taget til tiden), and 'Oversigt over poser' (475 Poser). On the right side, there is a 'Historik' (History) section showing a list of events with timestamps and descriptions. At the bottom right, there is a box for 'Enheds id' (Unit ID) showing 'Smila-00401'.

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The screenshot shows the SMILA web application interface. At the top, there's a header with the SMILA logo, the date 'Tirsdag 25.04.2023', and a user profile icon with 'Log off' and 'Danish' options. Below the header is a navigation bar with tabs: 'Overvågning', 'Enheder', 'Administration', 'Systemstyring', and 'Kommunikation'. The main content area is titled 'Medicinplaner og dosisruller'. It includes a 'MEDICINPLANER OVERSICHT' button and two buttons for 'FORRIGE BORGER' and 'NÆSTE BORGER'. Below these are fields for 'Navn: xxx xxx', 'Borger-ID: 501', 'Fødselsdato: 01.01.1901', and 'Dosisapotek: Pharmaservice Code128'. The 'Dosisruller' section has a search bar and buttons: 'TILFØJ DOSISRULLE', 'ADD POUCH ROLL WITH .CSV FILE', 'REDIGER DOSISRULLE', and 'SLET DOSISRULLE'. A table lists dosage rolls with columns: 'Rulle-ID', 'Startdato og klokkeslæt', 'Slutdato og klokkeslæt', 'Medicinaler', 'Dosisapotek dato', and 'Indført'. One row is visible with ID 'a38fc911-8886-415f-8b37-f9fba9956d29'. To the right, there's a 'Ugeskabelon for medicinen' section with a calendar view and an 'Oversigt over poser' section with a table for 'Hent dato og klokkeslæt', 'Pose-ID', 'Rulle indført', and 'Rulle skubbet ud'. The bottom of the interface shows pagination: 'Rækker pr. side: 10' and '1 af 1'.

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The image shows a confirmation dialog box with a large orange exclamation mark icon at the top. The text inside the dialog reads: 'Er du sikker??' followed by 'Er du sikker på, du ønsker at slette dosisrullen i perioden fra 24.08.2024 08:00 til 27.08.2024 16:00?'. At the bottom, there are two buttons: a blue button labeled 'Ja, slet det!' and a grey button labeled 'Annuller'.

Deleted

Pouch roll for period from 11.01.2024 06:00 till 24.01.2024 06:00 has been deleted successfully

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*EU Declaration of Conformity
(according to the Medical Device Regulation 2017/745 and the radio equipment directive
2014/53/EU)*

Hersteller <i>manufacturer</i>	JDM Innovation GmbH Carl-Benz-Str. 16 71711 Murr Germany
Produkt <i>product</i>	Medical Dispenser SMILA MD-1
Basis UDI-DI <i>Basic UDI-DI</i>	PP10483JDMI.04201901.1667

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
The Blister-Dispenser is intended to support and improve the medication adherence of individual or several patients. The Blister-Dispenser is designed for home-use as well as for the use in nursing homes or hospitals. The device is intended to be operated by the patient himself or a caregiver. The Blister-Dispenser can hold one or multiple patient individual medication blisters. Based on the medication plan the Blister-Dispenser delivers the pouches to the patient. Caregivers and service employees can monitor the medication adherence through the cloud-platform.

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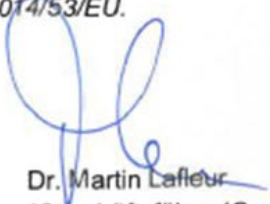We declare under our sole responsibility that the product mentioned above is in conformity with the General Safety and Performance Requirements of Annex I of the Medical Device Regulation 2017/745 and with the RED 2014/53/EU.

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0.3. Copyright